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### INDICAÇÕES

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## HALLUX VALGUS PROTECTION with Epithelium 26®

### INDICATIONS

To prevent pain caused by pressure or excessive friction on bunions.

### CONTRA-INDICATIONS

- Do not use on a skin lesion.
- Persons with arteritis, diabetes, polyneuritis, peripheral neuropathy, or foot ulcers: consult your physician or chiropodist before application and monitor your foot during use.

### PRECAUTIONS FOR USE

- Wash before first use.
- The hallux valgus protection pad should be worn next to the skin, on the right or left foot.
- Slip your foot into the protection pad, putting your big toe in the tubular part. The Epithelium 26® gel module should be placed on the hallux valgus.
- To help the protection pad to remain in position, wear tights, socks or stockings over it.

### WHAT IT IS USED FOR

- Patented by chiropodists, Epithelium 26® is a silicone gel with similar pressure distribution properties to those of human subcutaneous tissue.
- For "bunions", the thin module of Epithelium 26® enables the hallux valgus protection to effectively minimize impingement pain but without additional pressure.

Hallux valgus protections are for protective purposes only; they are not designed to correct the bunion.

### CLEANING

- Machine-washable at 30°C in the washing net provided or hand-wash (without scrubbing).
- Leave to dry naturally. Do not iron.

**COMPOSITION:** 61% polyamide/elastane fabric, 39% silicone.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2001.

 Machine washable at 30°C.	 Do not iron or steam.	 Do not dry clean.
 Do not bleach.	 Do not tumble-dry.	 See packaging.

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